


Baby Gender Test

Instructions for Use

Step 1: Draw Blood


Perform a venipuncture blood draw using standard protocol. The tube provided contains an additive to help preserve the sample.

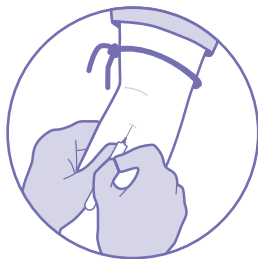

Instructions for Use

Step 2: Mix Solution

When blood draw is complete, mix thoroughly by inversion 10 times.

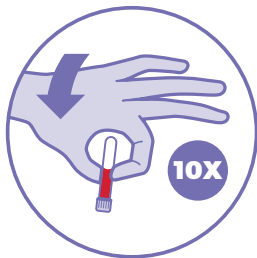

Step 3: Ensure Sample Viability

After collection, transport and store tubes at temperatures between 6° C and 37° C.

DO NOT FREEZE. Ship immediately or no later than 1 day after sample collection.

Not following these guidelines may result in the need for sample recollection and a delay in issuing the test result.



Step 4: Complete the Test Request Form

Complete and sign the **Test Request Form** included with your kit. Be sure to include all information including the kit barcode and your email address to receive the results report.

Missing or illegible information will delay your results.

Do not write on tube. The tube's barcode is all that is needed to identify the tested party.

Step 5: Pack and Ship

Place the collection tube into the kit's small resealable plastic bag and seal. Securely nestle bag with sample into the protective foam provided and place bag and foam in kit box.

Scan or photograph the completed Test Request Form and email it to us so we can book courier collection. Place the form and kit box in the FedEx Clinical Pak provided. Sign and date all copies of the commercial invoice enclosed with the waybill in the pouch on the front of the Pak and place back in the pouch.

We will arrange for Federal Express to collect the package and ship it directly to the laboratory.